EN

This medical device and Instructions for Use are intended for use by a health care professional.

Devce Description
Sorbact® Gel Dressing is a sterile, bacteria and fungi binding dressing covered with gel. It consists of a green Sorbact® wound contact layer with a water based gel containing carbomer and propylene glycol (10%). Sorbact® Gel Dressing donates moisture, enables a moist wound environment and facilitates autolytic debridement.

Mode of Action
Sorbact® binds microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus species*, *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown *in vitro*. These microorganisms are removed from the wound each time the dressing is changed.

Intended Use
Sorbact® Gel Dressing is intended for use in management of clean, contaminated, colonized or infected wounds with dry to low exudate levels, such as traumatic wounds, burns, cavity wounds, fistulas, pressure ulcers, diabetic ulcers and foot and leg ulcers.

Contraindications
There are no known contraindications to the use of Sorbact® Gel Dressing. Attention is drawn to the warnings and precautions.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect, e.g. skin irritation is observed, discontinue use. Do not use on patients with a known allergy to propylene glycol or to any other components of the dressing. Caution should be taken when exposing children below 5 years and patients suffering from liver and kidney disease, due to limited capacity to metabolize propylene glycol. High exposure, e.g. in case of simultaneous use of other products containing propylene glycol, may in rare cases lead to adverse events, e.g. hypersmolality or metabolic acidosis. The dressing should always be applied in direct contact with the wound area to allow microorganisms to bind to the dressing. Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not apply Sorbact® Gel Dressing on wound edges, to avoid maceration of the surrounding skin. Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of microorganisms.

Sorbact® Gel Dressing is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination.

Do not re-sterilize.

Do not use if peel pouch is damaged.

Prior to radiation therapy remove Sorbact® Gel Dressing. A new dressing can be applied following treatment.

Sorbact® Gel Dressing is MR safe.

Instructions for Use 1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound.
3. Remove the dressing from the pouch using an aseptic technique.
4. If cutting the dressing, use an aseptic technique. Discard any open and unused dressing.
5. Apply the dressing. Ensure that the dressing comes into direct contact with the complete wound surface. The dressing should not overlap the surrounding skin.
6. In cavity wounds and fistulas, always leave a part of the dressing sticking out so that it can be easily removed.
7. Apply an appropriate secondary dressing.
8. The dressing change frequency depends on exudate levels and overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

Storage and Disposal
Disposal should be made according to local environmental procedures.

Gel amount:

REF	98136	98137	98138FR	98139	98180	98181
gram/product	5.3	10	15	10	65	90

All products may not be available in all markets.

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