ENG **Hydrophobic microbe binding dressing pad**

Device Description

Cutimed® Sorbact® Dressing Pad is a sterile, hydrophobic microbe binding wound dressing. It consists of a green Sorbact® wound contact layer combined with an absorbent core and a white backing. Cutimed® Sorbact® Dressing Pad absorbs and retains exudate, thereby reducing the risk of maceration and enabling a moist wound environment. The dressing can be used in compression therapy.

Mode of Action

Cutimed® Sorbact® can bind hydrophobic microbes based on in vitro testing of the following species: S. aureus including MRSA, S. gordonii, P. aeruginosa, and E. coli. When the dressing is removed, microbes adhered to the dressing will be removed.

Intended Use

Cutimed® Sorbact® Dressing Pad is indicated for use in the management of exuding partial to full thickness wounds (including clean, contaminated, colonized or infected wounds): post-operative wounds, trauma wounds, pressure ulcers, diabetic ulcers, and venous ulcers.

Contraindications

There are no known contraindications to the use of Cutimed® Sorbact® Dressing Pad. The Warnings and Precautions should be noted.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect is observed, consult a physician or an appropriate health care provider.

Do not use on patients with a known hypersensitivity to any of the dressing components.

The green wound contact layer should always be applied in direct contact with the wound area to allow hydrophobic microbes to bind to the dressing.

Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of hydrophobic microbes.

Cutimed® Sorbact® Dressing Pad is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross-contamination.

Do not re-sterilize.

Do not use if the peel pouch is damaged.

Do not cut the dressing.

Prior to radiation therapy, remove Cutimed® Sorbact® Dressing Pad. A new dressing can be applied following treatment.

Cutimed® Sorbact® Dressing Pad is magnetic resonance imaging (MRI) safe.

Instructions for Use

1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound. The wound pad should overlap the wound margins by at least 2 cm.
3. Remove the dressing from the pouch using an aseptic technique.
4. Apply the dressing. Ensure that the green contact layer comes into direct contact with the complete wound surface.
5. Apply an appropriate dressing fixation.
6. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Depending on the clinical condition, the dressing can be left in place for up to 7 days.

Storage and Disposal

Keep dry and away from sunlight.

Dispose of waste according to local environmental procedures.

Caution

Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

Cutimed® is a registered trademark of BSN medical GmbH.
Sorbact® is a registered trademark of ABIGO Medical AB.

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Made in Sweden
® = registered trademark

STERILE R

Sterilized using irradiation

Patent information:
www.essity.com/patents

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