ENG

Hydrophobic microbe binding dressing with hydrogel

Device Description

Cutimed® Sorbact® Gel Dressing is a sterile, hydrophobic microbe binding dressing covered with gel. It consists of a green Sorbact® wound contact layer with a water-based gel containing carbomer and propylene glycol (10%). Cutimed® Sorbact® Gel Dressing donates moisture, enables a moist wound environment and facilitates autolytic debridement.

Mode of Action

Cutimed® Sorbact® can bind hydrophobic microbes based on in vitro testing of the following species: *S. aureus* including MRSA, *S. gordonii*, *P. aeruginosa*, and *E. coli*. When the dressing is removed, microbes adhered to the dressing will be removed.

Intended Use

Cutimed® Sorbact® Gel Dressing is indicated for use in the management of dry to low exuding partial to full thickness wounds (including clean, contaminated, colonized or infected wounds): post-operative wounds, trauma wounds, shallow cavity wounds, fistulas, pressure ulcers, diabetic ulcers, and venous ulcers.

Contraindications

There are no known contraindications to the use of Cutimed® Sorbact® Gel Dressing. The warnings and precautions should be noted.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or if a side effect, e.g. skin irritation is observed, discontinue use.

Do not use on patients with a known allergy to propylene glycol or to any other components of the dressing. Caution should be taken when exposing children below 5 years and patients suffering from liver and kidney disease, due to limited capacity to metabolize propylene glycol.

High exposure, e.g. in case of simultaneous use of other products containing propylene glycol, may in rare cases lead to adverse events, e.g. hyperosmolality or metabolic acidosis.

The dressing should always be applied in direct contact with the wound area to allow hydrophobic microbes to bind to the dressing. Should the dressing dry or adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not apply Cutimed® Sorbact® Gel Dressing on wound edges, to avoid maceration of the surrounding skin.

Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of hydrophobic microbes.

Cutimed® Sorbact® Gel Dressing is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross-contamination.

Do not re-sterilize.

Do not use if the peel pouch is damaged.

Prior to radiation therapy, remove Cutimed® Sorbact® Gel Dressing. A new dressing can be applied following treatment.

Cutimed® Sorbact® Gel is magnetic resonance imaging (MRI) safe.

Instructions for use

1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound.
3. Remove the dressing from the pouch using an aseptic technique.
4. If cutting the dressing, use an aseptic technique. Discard any open and unused dressing.
5. Apply the dressing. Ensure that the dressing comes into direct contact with the complete wound surface. The dressing should not overlap the surrounding skin.
6. In cavity wounds and fistulas, always leave a part of the dressing protruding so that it can be easily removed.
7. Apply an appropriate secondary dressing.
8. The dressing change frequency depends on exudate levels and overall condition of the wound and surrounding skin. Depending on the clinical condition, the dressing can be left in place for up to 7 days.

Storage and Disposal

Dispose of waste according to local environmental procedures.

Caution

Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

Gel amount:

Size	7,5 cm x 7,5 cm 3 in x 3 in	7,5 cm x 15 cm 3 in x 6 in	15 cm x 20 cm 6 in x 8 in
gram/ product	5,3	10	65

Cutimed® is a registered trademark of BSN medical GmbH.
Sorbact® is a registered trademark of ABIGO Medical AB.

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Made in Sweden
® = registered trademark

STERILE R

Sterilized using irradiation

Patent information:
www.essity.com/patents

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